Operating instructions for electric cosmetic chairs

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## Luxoria PRO 5

Please read the manual and the included safety instructions before using the chair.

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PL POLSKI

strona 16

This manual provides detailed guidance on the operation and maintenance of the equipment. Before use, read this document carefully and follow the recommendations it contains. Keep these instructions for future reference. Ensure that all potential users of the equipment are familiar with the contents of this manual before using the equipment.

## I. INTRODUCTION

The Luxoria PRO 5 armchair is an excellent choice for beauty salons. The versatility of this product also makes it ideal for massage, pedology and physiotherapy practices, luxury wellness & spa centres and creative tattoo studios. With its robust construction and aesthetic design, the Luxoria PRO 5 armchair blends in perfectly with modern interior furnishings. Ergonomic design and advanced adjustment features ensure maximum comfort for clients and professionals alike.

## II. INSTALLATION

### **Unpack the chair:**

- Carefully open the package to avoid damaging the contents.
- Remove all packaging materials such as foil, foam or tape.
- Ensure that all seat components are present as per the contents list.

### **Check seat for damage:**

- Carry out a thorough visual inspection of the seat, paying particular attention to metal parts, plastic parts and upholstery.
- If you notice any scratches, dents, cracks or other damage, do not plug the chair.

### **Contact seller in case of damage:**

- Contact the seller by telephone or e-mail, informing them of any damage you have noticed.
- Include photos of the damage and packaging to help identify the problem.
- Follow the dealer's instructions on next steps, such as returning the seat or replacing parts.

### **Check completeness of accessories:**

- Ensure that all accessories listed in the instructions are present.
- If you find discrepancies with your order during delivery, please notify the seller immediately.

### **Storage and use of the electric control chair:**

- The chair should be stored and used in normal dry conditions, indoors, where the temperature is controlled between +10°C and +40°C and the relative humidity is between 30% and 75%.
- If the chair will be stored at temperatures below 0°C, set the recommended room temperature for at least 5 hours before using it.

### TIPS!

- Children and inexperienced people should not use this electric cosmetic chair.
- Please read the operating instructions carefully before use.
- Children must be supervised and should not play near the seat to avoid risks.
- For safety, the electric beauty chair and its systems should be switched off when not in use and left unattended.
- After use, unplug the power cord.

### III. CONNECTION TO MAINS

Before inserting the plug into an electrical outlet, ensure that:

- The socket is connected correctly to earthing and that it complies with current regulations.
- The socket is able to withstand the maximum power load of the appliance.
- The socket corresponds to the plug. If not, the socket must be replaced.
- The mains voltage is as specified in the Technical Data table.


- Do not use extension cords.
- Use an unbent cable.
- The electrical supply cable may only be replaced by an authorised specialist.
- Do not carry out any repairs to electric actuators yourself.
- Avoid direct sunlight, UV radiation, rain, excessive dust, humidity.

**NOTE!**

The manufacturer is not responsible if the above rules are not followed.


### IV. LUXORIA PRO 5 ELECTRIC CHAIR CONNECTION AND OPERATING INSTRUCTIONS

Connect the power cord to a suitable external power source, ranging from 110 V to 240 V.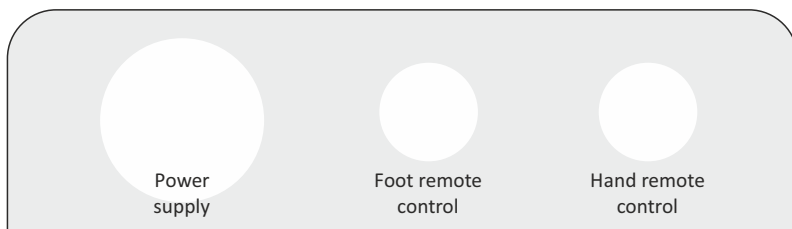



**Armrest installation method:**

The chair must be in a horizontal position in order to fit armrests to it. Slide the armrest into the hole located on the side of the backrest. The armrest should be firmly attached. Twist it so that it goes into place properly. Do the same with the other armrest. The armrest can be rotated by 180°.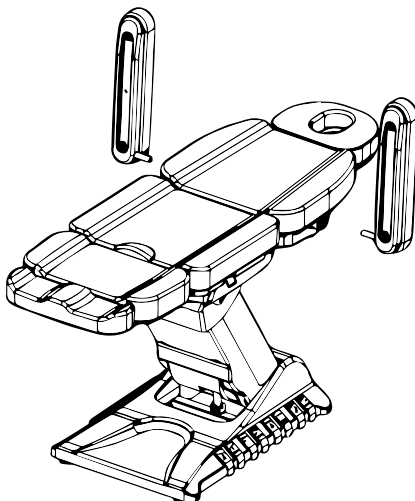



The Luxoria PRO 5 cosmetic chair is fully controllable by electric remote control.

**A-B:** Electrically adjustable seat height (66-94 cm).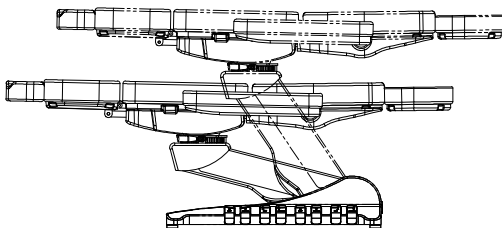
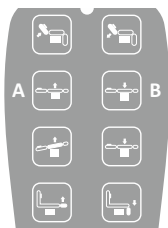


**C-D:** Electrically adjustable back support angle (0-80°).

**E-F:** Footrest angle adjustment (0-90°).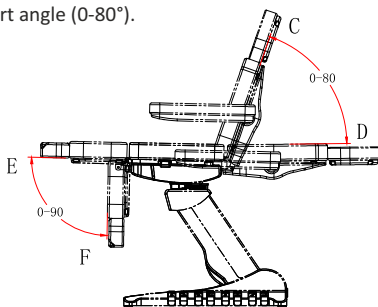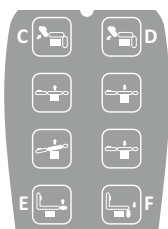

**G-H:** Angle adjustment of the entire chair (0-15°).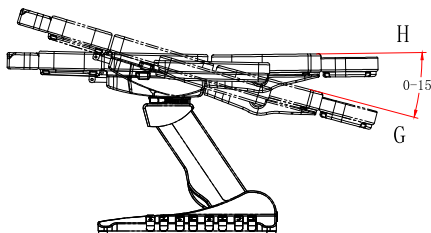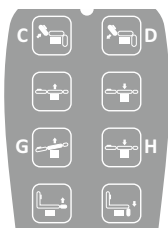

**Additional settings on the remote control:**

**J:** Reset settings: backrest raised to highest level, legs lowered to lowest level and entire seat to lowest level.


**K:** Reset settings: the backrest lowers to the lowest level and the legs raise to the highest level.

**L:** Switching the backlight on/off.


**M:** Lock, hold button for one second to lock, all functions, other key operations disabled; hold for 3 seconds to unlock.

**N:** Heating on/off.

**P:** Display of backrest and seat temperature in real time. When you turn the heating on, the '+' and '-' buttons are used to adjust the temperature.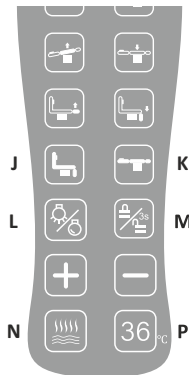

## Control function via buttons integrated in the product frame:

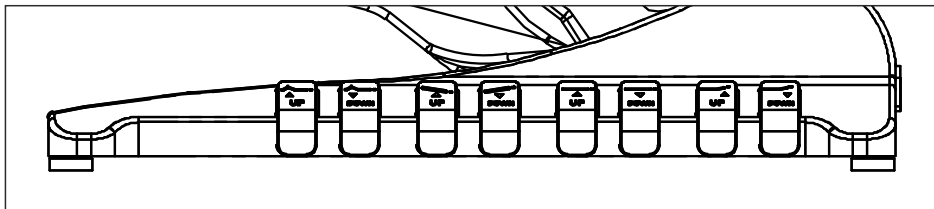
1. Electrically adjustable angle of footrest adjustment (0-90°).
2. Electrically adjustable angle of inclination of the entire seat (0-15°).
3. Electrically adjustable seat height (66-94 cm).
4. Electrically adjustable back support angle (0-80°).

### Additional chair settings possible.


#### Footrest/headrest extension function:

In order to adapt the product to users of different heights, the footrest and headrest have been designed with an adjustable length, allowing it to be extended for optimum comfort and support.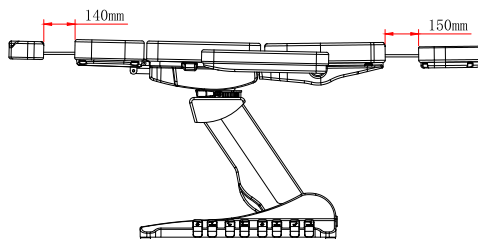

#### Chair rotation function:

To enable the chair to rotate from 0° to 120°, simply release the lever located under the seat. This gives the user complete freedom of movement and unparalleled comfort and convenience.

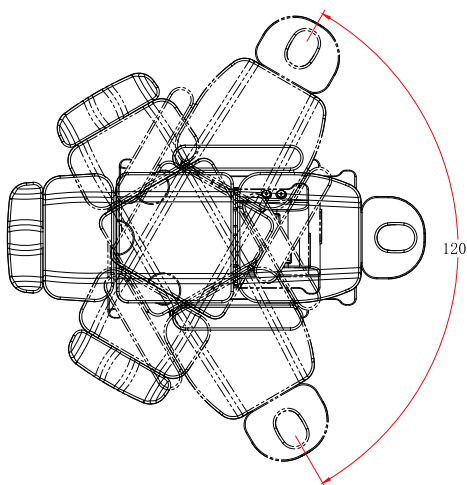## V. WARNINGS AND PRECAUTIONS

- Use the chair in accordance with the operating instructions.
- Damage caused by non-compliance with the above remarks does not constitute grounds for complaint.
- This appliance is not intended for use by persons with reduced physical or mental capacity or lacking experience and knowledge, unless they use it under supervision or after receiving instructions from a person responsible for their safety.
- The chairs should only be used by adults and in accordance with the instructions.

- The chair is designed for indoor use in dry and confined spaces and for treatments carried out exclusively for one person at a time.
- Do not move the chair with the patient.
- It is forbidden to sit down abruptly on the seat (jumping in).
- Sitting, kneeling and leaning on the bolster, shelves and armrests is prohibited.
- Avoid prolonged contact of the seat upholstery with oils. Oilproof coverings or waterproof sheets should be used for this purpose.
- The chair should be used and stored away from direct sources of fire and heat.
- Do not expose the seat to prolonged sunlight to avoid discolouration of the upholstery.
- After bringing the chair from the cold room, wait 15 minutes before plugging it in.
- It is advisable to fully adjust the height of the seat from minimum to maximum to ensure even distribution of the lubricant inside the engine. This should be repeated 2-4 times a week to ensure proper maintenance of the seat and to prolong the life of the engine.
- Do not remove the plug from the socket by pulling on the cable.
- Do not dismantle the unit or attempt to access its interior.
- When getting on and off the chair, special caution must be exercised. Do not sit on the edges of the seat or place your weight on elements such as the footrest, armrests, or headrest. Such behavior may cause the chair to lose stability, which can result in the user falling and the device being damaged.
- The manufacturer is not liable for accidents, injuries, or damages resulting from improper use of the chair, particularly from failure to follow the safety instructions described in this manual.
- The chair may only be used if the above recommendations are met.

## VI. MAINTENANCE AND MAINTENANCE MEASURES

### **\*Unplug the power cord before cleaning\***

To keep the surface of the electric cosmetic chair in impeccable condition, it must be cleaned regularly. It is important to carry out cleaning and disinfection before each new user to ensure hygiene and comfort. In addition, thorough disinfection is recommended every month to maintain the highest standards of cleanliness and safety.

### **Metal and plastic surfaces**

To keep metal and plastic surfaces and remote controls in pristine condition, use a soft cloth and a gentle, cleaning agent. Use a small brush to clean hard-to-reach areas such as corners and other nooks and crannies. When cleaning, wipe the surfaces with a cloth, avoiding excessive amounts of liquid. Then use a disinfectant, following the manufacturer's instructions, and let the surfaces dry to room temperature.

### **Chair surface**

To keep the chair's surface in perfect condition, wipe it down with a clean, soft cloth and then evenly apply a special upholstery cleaner. After its application, wipe the surface again with a clean, soft cloth to ensure even cleaning. Regular cleaning has a positive effect on the quality and life of the surface.

### **Maintenance:**

Professional users are responsible for implementing regular maintenance procedures, both monthly and semi-annually. Inspect the controls and their wiring weekly, checking the condition and function of the following parts at least once every six months:

1. Power cable and its attachment.
2. Engine wiring.
3. Correct attachment of accessories.
4. Complete all adjustments and make sure the bed is working properly.


If any defects are found, stop using the equipment immediately. For example, if the device makes unusual noises or does not work properly, contact the after-sales service. Only experienced or authorised personnel may open or replace the actuator or control unit.

!

In accordance with the provisions of the law on electrical and electronic equipment waste, it is prohibited to dispose of used equipment marked with the symbol of the crossed out bin with other household waste. Used equipment should be returned to proper collection point. The above statutory obligations were introduced in order to limit the amount of waste generated from electrical and electronic equipment waste and ensure an appropriate level of collection, recovery and recycling of used equipment. The correct implementation of these obligations is important especially when used equipment contains hazardous components that have especially negative impact on the environment and human health.

## VII. PRODUCT SPECIFICATIONS

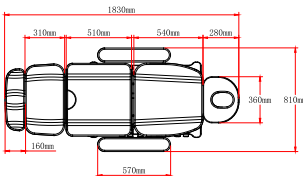	Dimensions	Height adjustment range	Weight	Maximum load	Temperature control	Electricity supply
Luxoria PRO 5		66 - 94 cm	66 kg	200 kg	Heat level between 35°C and 60°C.	110V-240V

## VIII. MEANING OF PICTOGRAPHS

Attention

Keep dry during storage

See the instruction manual for use

Maximum load

CE mark

## VIII. WARRANTY CONDITIONS

1. For the purposes of the guarantee:
  - a) "Manufacturer" refers to HABYS Spółka z ograniczoną odpowiedzialnością (Limited Liability Company) with its registered seat in Jasło, ul. Produkcyjna 16, 38-200 Jasło, entered into the Register of Entrepreneurs kept by the District Court in Rzeszów, XII Commercial Division of the National Court Register, with the assigned KRS number: 0000513317, with the share capital in the amount of PLN 10.750.000 (ten million seven hundred fifty thousand zlotys), the capital has been fully paid up, NIP: 6852208438, REGON: 180186290.
  - b) "Buyer" refers to the final Buyer of the equipment, manufactured by the Manufacturer, including both the consumer and the entrepreneur.
  - c) "Distributor" refers to an entity that, under an agreement with the Manufacturer, sells the Equipment in a designated area.
  - d) "Authorized Service Centre" refers only to a service held by the Manufacturer or a service indicated by the Manufacturer and held by its Distributor,
  - e) "Equipment" refers to a movable item, manufactured by the Manufacturer, being the subject of the sales and to which the said warranty applies.
2. **The manufacturer ensures good quality and efficient operation of the equipment, for which the warranty card has been issued, for the period of: 2 years for constructional elements, 2 years for upholstered elements and 2 years for electronic and hydraulic accessories.**
3. The guarantee repair does not include the activities provided for in the manual, which the Buyer is obliged to perform on his own (installation of the Equipment, maintenance of the Equipment, etc.).
4. Liability under the guarantee covers only defects resulting from causes inherent in the Equipment. In particular, the guarantee does not cover any damage resulting from improper or inconsistent use, storage, maintenance or transport of the Equipment, as well as colour differences between the parts exposed and not exposed to solar radiation, the influence of chemical or abrasive agents, atmospheric pollution or accidental punctures and abrasions, the presence of stains or rings from greasy or dyeing substances (fats, creams, lotions or oils), permanent staining with dyes such as inks, permanent markers or other non-permanent dyes used in the production of clothing (e.g. dyes used to make blue jeans).
5. The guarantee does not cover any mechanical damage to the Equipment.
6. The buyer loses the guarantee rights in the event of:
  - a) the Manufacturer states that the Equipment was repaired by entity other than the Authorized Service,
  - b) making any design modifications to the Equipment without the prior written consent of the Manufacturer,
  - c) using the Equipment contrary to its intended use,
  - d) using the Equipment after revealing any defect in it.
7. Regardless of the loss of the guarantee rights, described above, in point 6, the Manufacturer shall not be liable for any damage to property or caused to third parties, which may be caused by the use of the Equipment covered by the guarantee.
8. The guarantee does not exclude or limit the rights resulting from the warranty for defects. Exercising the rights under the guarantee does not affect the seller's liability under the warranty for defects. If the Buyer exercises the rights under the guarantee, the time limit for the exercise of the rights under the warranty for defects is suspended on the date the Manufacturer is notified of the defect. The period runs further from the date of the Manufacturer's refusal to perform the obligations arising from the guarantee or the ineffective expiry of the time for their performance.
9. The scope of the guarantee coverage is limited to the territory of the country in which the Equipment was sold.
10. The Manufacturer shall perform its obligations under the guarantee (removal of defects or damage to the structure of the Equipment arising from reasons inherent in the Equipment, and disclosed during the guarantee period) free of charge within 14 days from the date of delivery of the Equipment to the Manufacturer or the Authorized Service Centre. If the repair of the Equipment cannot be performed within the above-mentioned period, the Manufacturer shall inform the Buyer of the above, and also indicate the reasons for the delay and the expected date of fulfilment of the recognized guarantee request.
11. The use of the guarantee rights requires the total (cumulative) fulfilment of four conditions:
  - a) presenting a VAT invoice, receipt or other proof of purchase of the equipment to the Manufacturer, photographic documentation of the defective Equipment, together with a brief description of the reasons for submitting a guarantee claim,
  - b) reporting the defect within 14 (fourteen) days of its detection,
  - c) delivering the Equipment in original packaging at the expense of the Manufacturer, to the address indicated above in the guarantee card or to the address of the Authorized Service Centre,
  - d) designation of the place where the Manufacturer delivers the equipment after repairing it (limited to the country in which the Equipment was purchased).
12. The guarantee period is extended for the duration of the repair, starting from the date of delivery of the Equipment to the Manufacturer or to the Authorized Service Centre, to the date on which the Equipment was returned to the Buyer.
13. The responsibility of the Manufacturer or the Authorized Service Centre is to define the scope and the method of repairing the Equipment.
14. The guarantee is limited to the repair of the Equipment. The Manufacturer reserves the right to replace the Equipment with a non-defective one, if the repair exceeds the price of the Equipment indicated in the proof of purchase. Under the guarantee, the Buyer shall not be entitled to demand that the Equipment be replaced with a new one. The guarantee, in particular, does not cover liability for losses or damages incurred by the Buyer in connection with the fact that one is not able to use the Equipment.
15. The Manufacturer shall not be liable for any damage to the Equipment caused during the performance of the transport service by the delivery company until it is collected by the Manufacturer from the delivery company.
16. If the shipment of the Equipment was ordered by the Manufacturer, the Buyer is obliged to check the technical condition of the Equipment upon its receipt. If the Buyer finds that the Equipment has been damaged during the transport, the Buyer is obliged to:
  - a) immediately, not later than within 3 days from the date of receipt of the shipment, notify the Manufacturer of any damage to the Equipment that occurred during transport, in order to enable the Manufacturer to submit a complaint to the delivery company and pursue related claims,
  - b) provide the Producer with a complaint report drawn up between the Buyer and the delivery company.
17. The guarantee regulations contained in this document are the sole and exclusive guarantee rules for HABYS limited liability products.

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