

IV. Placing the patient

The patient sits on the seat (3), puts his knees on Knees Rest (4), leans forward and rests hands on the Front Shelf (5) and puts his face in the pillow of regulated Face Rest (7).

V. Maintenance and cleaning

Chromium-plated and lacquered elements. For cleaning such elements products that do not contain abrasives are recommended. Exemplary disinfectants: Incidur Spray, Incidin Foam Spray, tissues Sani Cloth Active.

VI. Disinfection

Disinfect when necessary and sporadically.

VII. Operational remarks

1. Before starting the treatment, check the correct position of the frame as well as the technical condition and tension of the cable.
2. The chair is intended for the procedure of one person exclusively.
3. The chair cannot be moved with the patient.
4. Sudden sitting or jumping on the chair is forbidden.
5. Do not adjust the height of the gas spring when the chair is fully loaded.
6. When using the chair no element of Front Shelf (5) can be lean on the chair's frame.
7. Face or chest cushion should not be used as a handle.
8. The chair should be used in accordance with the instruction manual.
9. Every person using the couch should familiarise oneself with the terms of the warranty and to observe them.

VIII. Technical details

| | |
|------------------------------------|--------------|
| With of the base | 50 cm |
| Length of the base | 63 - 78 cm |
| Height with the Face Rest | 110 - 137 cm |
| Height from the ground to the seat | 55 - 61 |
| Weight without accessories | 9 kg |
| Maximum load allowed | 150 kg |

INSTRUCTION MANUAL

for the rehabilitation chair



Prestige-reh






HABYS Sp. z o.o.
38-200 Jasło, ul. Produkcyjna 16, Poland
tel.: +48 13 44 62 788, serwis@habys.pl

www.habys.com

I. Application

Rehabilitation Prestige-Reh chair is used for the treatment or soothing the course of a disease and the effects of injuries and handicap by allowing to perform effective and safe rehabilitation and therapeutic massages.


II. Design

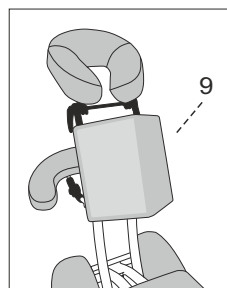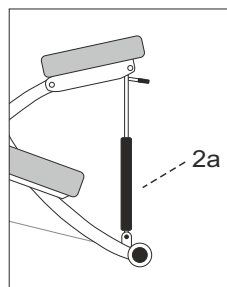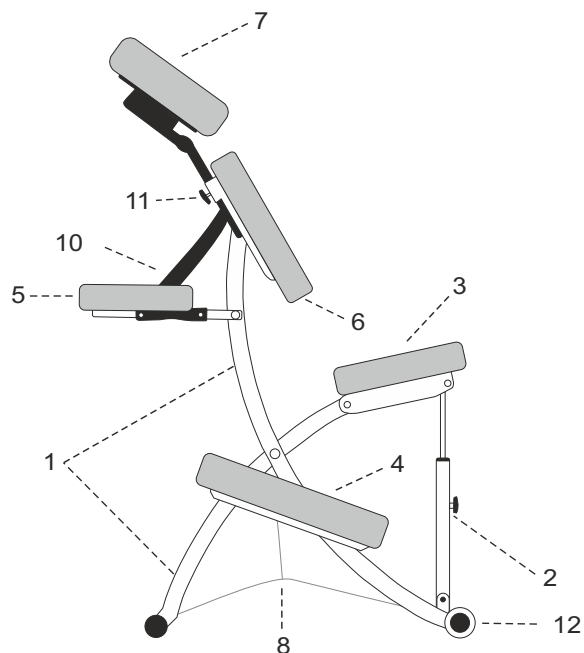

1. Frame
2. Telescopic regulated height
- 2a. Regulated gas-lifted height (option)
3. Upholstered Seat
4. Upholstered Knee Rests
5. Upholstered Front Shelf
6. Upholstered Chest Pillow
7. Adjustable Face Rest
8. Change of the height line
9. Sternum Pad (option)
10. Regulating the location of shelf bar
11. Knobs adjusting the height of the face rest
12. Transport wheels

III. Operation

1. Unfolding, folding and transport of the chair.

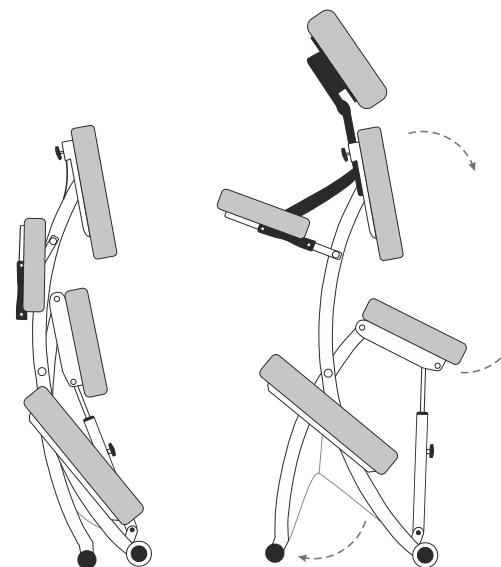

Place one hand on the chest pillow (6) and the other on the seat (3), open the chair by spreading the base of the frame on the sides.

Fold similarly, paying attention that the base of the frame / legs come into contact with each other. Incorrect folding of the chair prevents from inserting it into the carry case.

The chair can be transported in carry case or through the engagement of the belt conveying directly into the frame of the chair.

2. Adjustment of the height of the chair and the angle of the seat

The height is adjusted by means of a height adjuster located on a steel cable (8) and the angle of the seat (3) by means of a gas spring (2a) or a telescopic height adjustment (2).

3. Adjusting the position of the front shelf (5)

Press the 2 opposite buttons under the shelf and set it in the required position. To level it, the mounting strap must be lengthened or shortened.

4. Face rest adjustment (7)

The adjustable face rest (7) is attached to the chair with two wing knobs. To move the face rest, unscrew both knobs, set the face rest to the required height, and then tighten the knobs again.

5. Sternum pad (9) (optional)

Recommended for pregnant women or those with large breasts. In order to assemble it, detach the chest pillow (6) and replace it with the sternum pad by means of Velcro.